QUANTITATIVE DETERMINATION OF PLASMA PROTEINS

IVD

Store at 2-8°C

PRODUCT CHARACTERISTICS

The **PROT CAL REACTIVA SEARCH** is a liquid serum from human origin containing protein concentrations suitable to calibrate determinations of the constituents listed below by immunoturbidimetry method.

REAGENTS

PROT-CAL REACTIVA SEARCH	Calibrator. Human serum. Sodium azide 0.95 g/L. Concentrations of the constituents are stated on the list below.
---------------------------------	--

PRECAUTIONS

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However, handle cautiously as potentially infectious.

CALIBRATION

The concentration values of the Calibrator are traceable to Standard Reference Material ERM-DA470k/IFCC (all constituents except antithrombin III).

STORAGE AND STABILITY

The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.
Opened vials, stable for 30 days at 2-8°C.

PROCEDURE

To be used in immunoturbidimetric assays.

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